

E Collar Template

Courtesy of Hope for Gliders

Irving TX 75060

<http://hopeforgliders.org>

This the actual size of the collar we use. You may have to trim it to fit properly. I will overlap about 3/4 of an inch. We then fasten it together with three staples. It will probably take two people to get it on.

The e-collar should be made of thick report cover which is a sturdy plastic. The moleskin goes around the neck. Do NOT apply without the moleskin as it will cut into the gliders neck. The e-collar should be removed daily to check the neck for irritation and moleskin replaced if compressed or dirty. If you would rather invest in some good industrial strength Velcro you can staple that to the collar and it will be much easier to adjust and put on.

Be sure that the collar can turn when applied but not slip over the gliders head. Watch glider's nose for bluish or purple coloring or any difficulty in breathing, if observed then remove and reapply.

A big THANK YOU TO Mary & Charlie H for sharing this e-collar template.

